

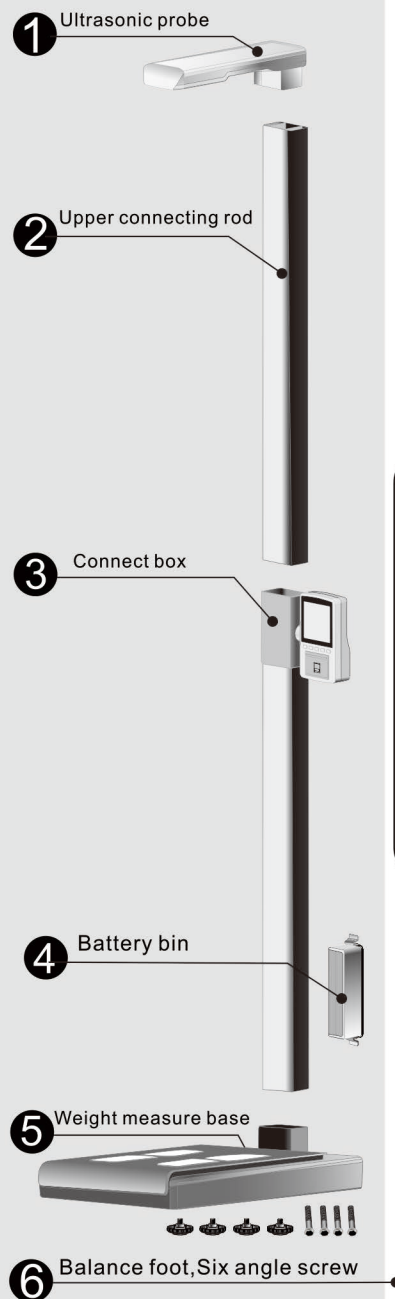


Installation Diagram

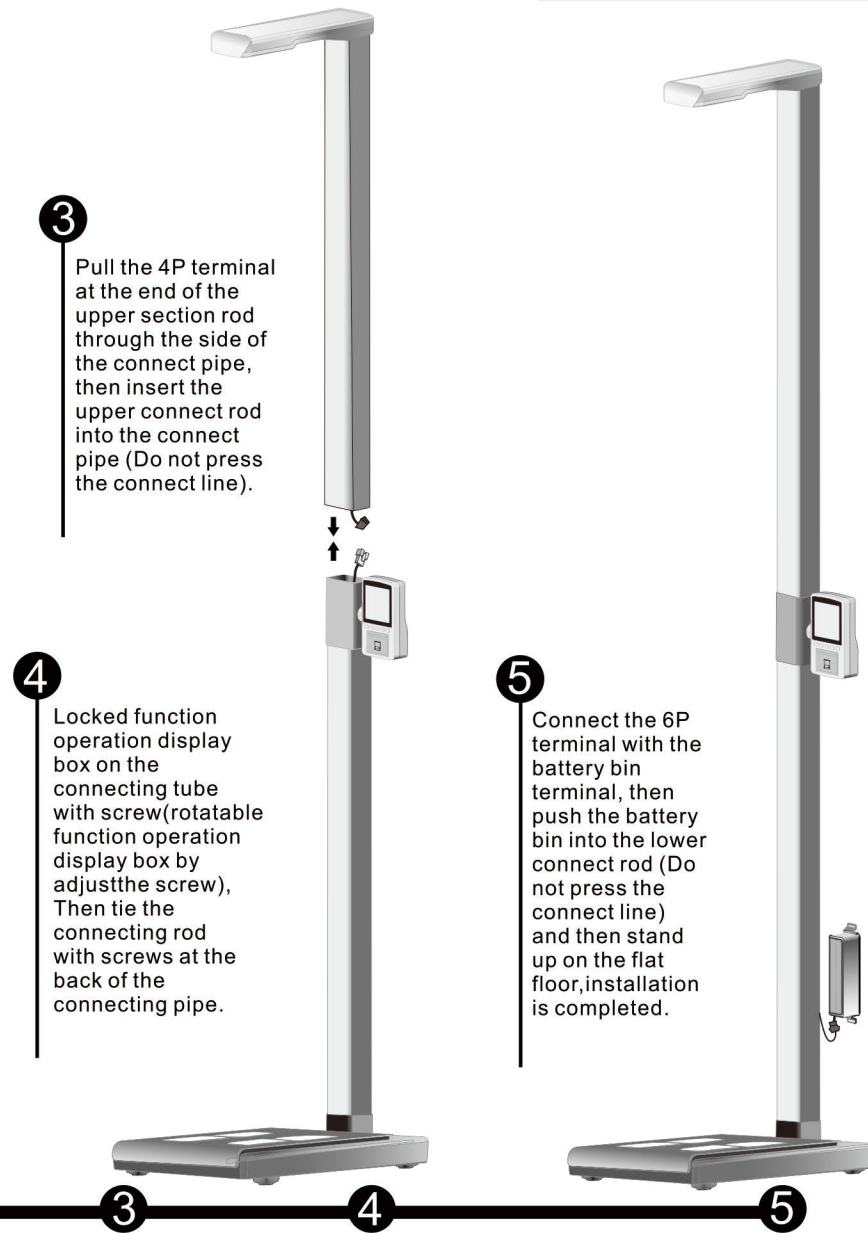

Package components

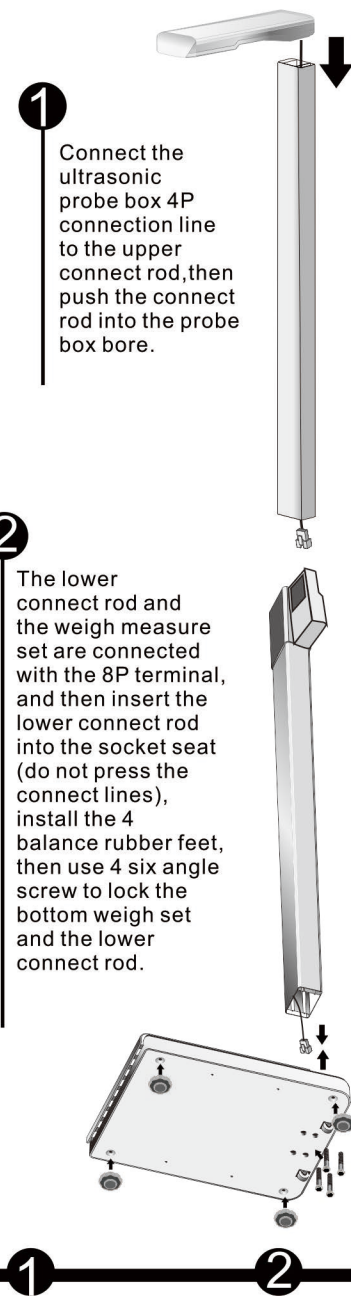

Function

- ◆ Weigh range:0.5kg-200kg (Weight division:0.05kg).
- ◆ Heigh range:80-210cm(Height division:0.2cm).
- ◆ Pressure start range :over 5kg.
- ◆ Display: Color display.
- ◆ With RS232 port,can connect with computer.
- ◆ Voice broadcast function.
- ◆ Measure height、 weight、 bodyfat、 water、 muscle、 bone Calorie、 BMI、 Figure.
- ◆ Measure the height with Ultrasonic.
- ◆ With printer,print out the measure report.
- ◆ Unit:kg/lb

Gender & AGE Setting

- ◆ After selecting the height, weight, and fat pattern, enter the sex setting, when the male icon flashes and goes in at the same time Enter the voice broadcast: "please enter your gender". Press or key to select gender. Press key to confirm.
- ◆ After the sex setting is complete, enter the age setting, when the age icon flashes, while the voice broadcast: "please enter your age", press or key to select age, press button to confirm.


Date and time setting

- ◆ Turn on the power, Enter the date setting status, set the lights flashing, press or key to select the date value, the key confirmation. (date set XXXX,XX, XX days).
- ◆ When the date setting is complete, enter the time setting status, set the lights flashing, press or select the time value, press the button to confirm (time setting xx:xx).

Error indication


- ◆ If the weight error, the LCD with show
- ◆ If the height error, the LCD with show

HEIGHT AND WEIGH MODE

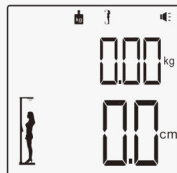

◆ The first use of the boot, default to this model, you can directly stand on the scales to measure, according to voice tips: please stand straight... measuring the height (you need to stand straight, otherwise it will affect the accuracy of the test) and you can leave for the measurement after listen the "drip" sound, then displays all the measured data, or press the print function to print out the data.


HEIGHT AND WEIGH ,BODY FAT MODE

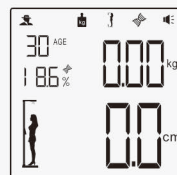

◆ Under this mode, you need to take off shoes and socks barefoot, stand on the 4 electrode test contact (each foot stand two electrodes) according to the voice tips: Please enter your gender, please enter your age, please stand on the scale, please stand up straight... Measuring your height (You need to stand straight to measure, otherwise it will affect the accuracy of the test). Please wait... (the machine is measuring your fat rate, you need to wait 2-3 seconds), then you can leave the measuring panel and wait for the measurement data. After the measurement, begin to broadcast the data (for example): your height is 170 centimeters, your weight is 59.05 kilograms, your fat is nine percent, the body shape is standard. All the measurement data is displayed on the screen, or press the print function to print out the data.

Notice of use

- ◆ Put the product on a flat floor before use.
- ◆ This product belongs to high precision electronic measurement products. In order to ensure its performance and service life, please do not place the product in the excessive damp and hot environment.
- ◆ Optimum use environment temperature 0-40°C If you do not use the product for a long time, please remove the battery from the battery bin.

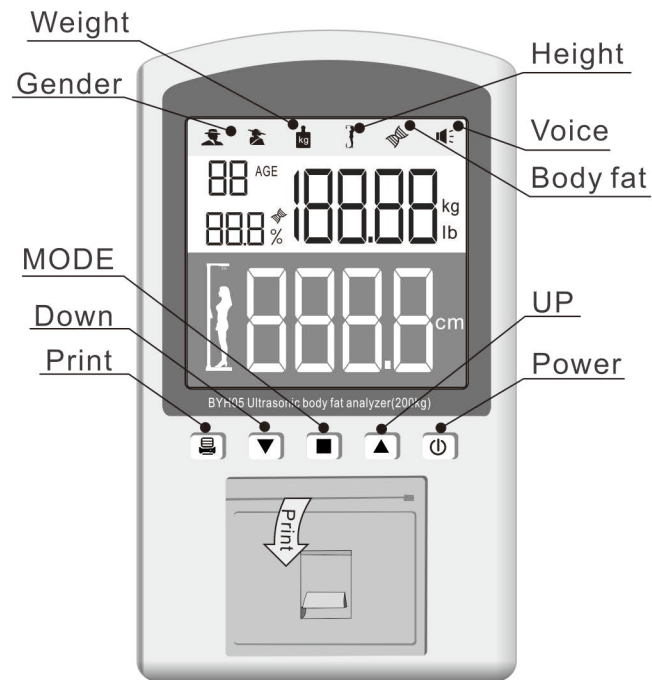

HEIGHT AND WEIGH MODE

May health be always with you	
Date	:20170808
Gender	:Male
Body Age	:30
Height	:170.6 cm
Weight	:68.65 kg (58-72)
BMI	:23.5 % (18-26)
Body Type	:Underfat (health)

HEIGHT AND WEIGH ,BODY FATMODE

May health be always with you	
Date	:20170808
Gender	:Male
Body Age	:30
Height	:170.6 cm
Weight	:68.65 kg (58-72)
BMI	:23.5 % (18-26)
Body Type	:Underfat (health)
Fat	:15.8 % (13-18)
Water	:59.6 % (52-63)
Muscle	:65.8 % (59-68)
Bone	:4.6 kg (3.8-5.1)
BMR	:1044 kcal (1005-1700)



Scan the WeChat

Get more health information at any time

(BYH05)

Ultrasonic body fat analyzer Instruction Manual



■ Please read the manual before use